

ATLANTIS™ screw torque recommendations for ATLANTIS™ abutments

The torque recommendations are a reference for your convenience.
They might change without further notice in order to follow manufacture's recommendations.

Manufacturer	Implant	Torque Ncm	Screwdriver
BioHorizons	BIOHORIZONS		
	Internal 3.5, 4.5, 5.7	30	0.050" (1.25 mm) Hex screwdriver
	Tapered Internal 3.5, 4.5, 5.7	30	
	External implant 3.5	30	
External implant 4.0, 5.0	30		
Biomet 3i	BIOMET 3i		
	Certain™ 3.25, 4.0, 4/3, 5.0, 5/4, 6.0	20	0.048" Hex screwdriver
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4, 5/6/5, 4/3, 5/4, 6/5	20	
	Certain™ XP 4/5, 5/6	20	
	Miniplant™ 3.25	20	
	Osseotite™ 3.75, 4.0, 5.0, 6.0	20	
Osseotite™ XP 4/5, 5/6	20		
Camlog	CAMLOG		
	CONELOG 3.3	20	1.28 mm Hex screwdriver
	CONELOG 3.8, 4.3	30	
	CONELOG 5.0	30	
	Screw-line Implant 3.3	20	
Screw-line / Rootline Implant 3.8, 4.3, 5.0, 6.0	20		
DENTSPLY Implants	DENTSPLY IMPLANTS		
	ANKYLOS® C/X – C/ 3.5, 4.5, 5.5, 7.0 (non-indexed)	15	1.0 mm Hex screwdriver
	ANKYLOS® C/X – /X 3.5, 4.5, 5.5, 7.0 (indexed)	15	
	OsseoSpeed™ EV 3.0 – Green	25	0.050" Hex screwdriver
	OsseoSpeed™ EV 3.6 – Purple	25	
	OsseoSpeed™ EV 4.2 – Yellow	25	
	OsseoSpeed™ EV 4.8 – Blue	25	
	OsseoSpeed™ EV 5.4 – Brown	25	
	OsseoSpeed™ Profile EV 4.2 Yellow	25	
	OsseoSpeed™ Profile EV 4.8 Blue	25	
	OsseoSpeed™ / OsseoSpeed™ TX 3.0 – Yellow	15	
	OsseoSpeed™ / OsseoSpeed™ TX 3.5, 4.0 – Aqua	20	
	OsseoSpeed™ / OsseoSpeed™ TX 4.5, 5.0 – Lilac	25	
	OsseoSpeed™ 4.0 - Blue (Previous)	25	
	OsseoSpeed™ TX Profile 4.5, 5.0	25	
	XIVE® S 3.0	24	
	XIVE® S 3.4, 3.8, 4.5, 5.5	24	
	FRIALIT™ 3.4, 3.8, 4.5, 5.5	24	
Keystone Dental	KEYSTONE DENTAL		
	PrimaConnex™ SD 3.3, 3.5, Genesis™ 3.8	30	Keystone Quad Driver
	PrimaConnex™ RD 4.0, 4.1, Genesis™ 4.5	30	
	PrimaConnex™ WD 5.0, Genesis™ 5.5, 6.5	30	

Please turn over

ATLANTIS™ screw torque recommendations for ATLANTIS™ abutments

Manufacturer	Implant	Torque Ncm	Screwdriver
Nobel Biocare	NOBEL BIOCARE		
	NobelActive™ 3.0	15	Nobel Biocare Unigrip
	NobelActive™/NobelReplace™ Conical Connection NP 3.5	35	
	NobelActive™/NobelReplace™ Conical Connection RP 4.3, 5.0	35	
	NobelReplace™ NP 3.5	35	
	NobelReplace™ RP 4.3	35	
	NobelReplace™ WP 5.0	35	
	NobelReplace™ 6.0	35	
	Brånemark System™ NP 3.3	35	
	Brånemark System™ RP 3.75, 4.0	35	
Brånemark System™ WP 5.0, 5.5	35		
Osstem	OSSTEM		
	Osstem TS Mini 3.5	20	1.2 mm Hex screwdriver
	Osstem TS Regular 4.0, 4.5, 5.0, 6.0, 7.0	30	
Straumann	STRAUMANN		
	Bone Level 3.3 NC (Narrow CrossFit™)	35	SCS screwdriver
	Bone Level 4.1, 4.8 RC (Regular CrossFit™)	35	
	Standard/Standard Plus 4.8 RN (Regular Neck)	35	
	Standard/Standard Plus 4.8 WN (Wide Neck)	35	
	Standard Plus 3.3 NN (Narrow Neck)	35	
	35		
Zimmer Dental	ZIMMER DENTAL		
	Tapered Screw-Vent 3.5, 4.5, 5.7 / Tapered Screw-Vent 3.5, 4.5, 5.7 friction fit	30	1.25 mm Hex screwdriver
	Screw-Vent 3.3, 3.7, 4.5 / Screw-Vent 3.3, 3.7, 4.5 friction fit	30	
	SwissPlus™ 4.8	35	
	35		

All trademarks and company names are the property of their respective owners.

Medical and legal disclaimer

The information (including torque recommendations) is provided for information purposes only and does not constitute advice for medical or treatment purposes, nor a legal contract, between DENTSPLY Implants and any person or entity. Although all reasonable effort is made to present accurate information (including torque recommendations), DENTSPLY Implants makes no guarantees of any kind, including but not limited to that it may be outdated, invalid or subject to debate. The information is provided "as is" without warranty of any kind, either express or implied, including without limitation, the implied warranties of merchantability, fitness for use of a particular purpose, or non-infringement.

You agree that neither DENTSPLY Implants, nor any other company within the DENTSPLY international group, will be liable to you for any loss or damages, either actual or consequential, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from DENTSPLY Implants. In particular, DENTSPLY Implants will have no liability for any consequential, indirect, punitive, special or incidental damages, whether foreseeable or unforeseeable, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from DENTSPLY Implants, whether based in contract, tort, statutory or other law, except only in the case of death or personal injury where and only to the extent that applicable law requires such liability.

